



**INSTRUCTIONS FOR USE**


Instrucciones de uso

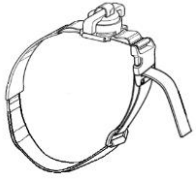


**1. Remove From Case**  
Quitar del caso

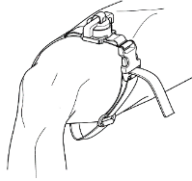


**2. Slide Over Limb**  
Deslizarse sobre la extremidad

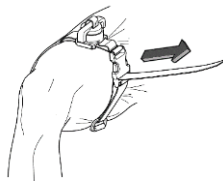

**3. Pull Strap Tight**  
Tirar de la correa apretada

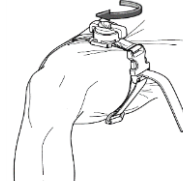


**4. Twist Handle Clockwise Until Blood Flow Ceases**  
Gire la manija en sentido horario hasta que cese el flujo de sangre

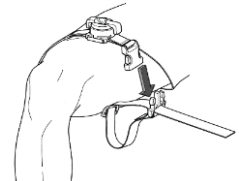


**5. Squeeze Buckle to Release**  
Apretar la hebilla para liberar

**INSTRUCTIONS FOR TRAPPED LIMB**


Instrucciones de extremidades atrapadas

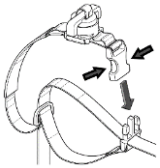

**1. Squeeze Quick Release Buckle and Separate**  
Apriete la hebilla de liberacion rapida y separe

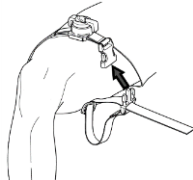


**2. Wrap Around Trapped Limb and Reconnect Quick Release Buckle**  
Envolver alrededor de la extremidad atarapada y Volver a conetar la hebilla

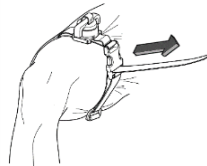

**3. Pull Strap Tight and Continue With Step 4 Above**  
Jale la correa y continue con el etapa 4 de arriba

**TOURNIQUET RESET INSTRUCTIONS**

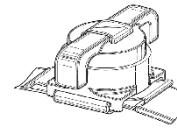


**1. To Reset the Tourniquet, Squeeze Quick Release Buckle. Hold the Base in One Hand and Lift Up On the Top to Disengage Teeth.**




**2. Turn Counter Clockwise Until All Twist Are Removed. Pull Straps From Both Sides to Reseat Handle**

**INSTRUCTIONS FOR USE**

Instrucciones de uso

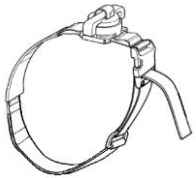


**1. Remove From Case**  
Quitar del caso

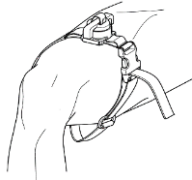


**2. Slide Over Limb**  
Deslizarse sobre la extremidad



**3. Pull Strap Tight**  
Tirar de la correa apretada

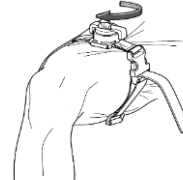


**4. Twist Handle Clockwise Until Blood Flow Ceases**  
Gire la manija en sentido horario hasta que cese el flujo de sangre

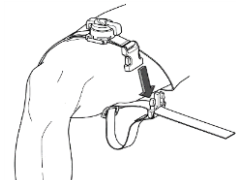

**5. Squeeze Buckle to Release**  
Apretar la hebilla para liberar

**INSTRUCTIONS FOR TRAPPED LIMB**

Instrucciones de extremidades atrapadas

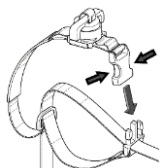


**1. Squeeze Quick Release Buckle and Separate**  
Apriete la hebilla de liberacion rapida y separe




**2. Wrap Around Trapped Limb and Reconnect Quick Release Buckle**  
Envolver alrededor de la extremidad atarapada y Volver a conetar la hebilla



**3. Pull Strap Tight and Continue With Step 4 Above**  
Jale la correa y continue con el paso 4 de arriba

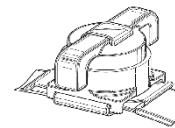

**1. To Reset the Tourniquet, Squeeze Quick Release Buckle. Hold the base in one hand and lift up on the top to disengage teeth.**

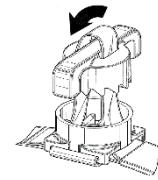

**2. Turn Counter Clockwise Until All Twist Are Removed. Pull Straps From Both Sides to Reseat Handle**

**LIMITATION OF LIABILITY**

RCR Medical Products, LLC, its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the X8T-T tourniquet. The user assumes all the risks of liability. The X8T-T should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician.

**STANDARD USE GUIDELINES**

When applied according to directions, the X8T-T is a safe and effective device for controlling life-threatening extremity bleeding. If you are unsure where the bleeding is coming from, the X8T-T can effectively be applied over clothing as high on the arm or leg as possible. The X8T-T must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the X8T-T repositioned 2"-3" above the injury, directly on the skin.

EC	REP
----	-----

Mdi Europa GmbH  
Langenhagener Str. 71  
D-30855, Langenhagen

Phone +49 (0)511 39089530  
Email: info@mdi-europa.com

View Instructional Videos at [www.RCRmedic.com](http://www.RCRmedic.com)

**LIMITATION OF LIABILITY**

RCR Medical Products, LLC, its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the X8T-T tourniquet. The user assumes all the risks of liability. The X8T-T should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician.

**STANDARD USE GUIDELINES**

When applied according to directions, the X8T-T is a safe and effective device for controlling life-threatening extremity bleeding. If you are unsure where the bleeding is coming from, the X8T-T can effectively be applied over clothing as high on the arm or leg as possible. The X8T-T must NOT be applied over solid objects within the clothing. As soon as the situation permits, the injured limb should be evaluated and the X8T-T repositioned 2"-3" above the injury, directly on the skin.

EC	REP
----	-----

Mdi Europa GmbH  
Langenhagener Str. 71  
D-30855, Langenhagen

Phone +49 (0)511 39089530  
Email: info@mdi-europa.com



**RCR MEDICAL PRODUCTS, LLC**

**X8T**  
tourniquet

**Tourniquet**



**RCR MEDICAL PRODUCTS, LLC**

**X8T**  
tourniquet

**Tourniquet**

